



**INTERNATIONAL CANINE SEMEN  
BANK, Inc. (ICSB) • MAIN OFFICE**  
P.O. BOX 651 • SANDY, OREGON 97055  
Phone: (503) 663-7031 Fax: (503)676-8025  
E-mail: ik9sb@aol.com • Website: www.ik9sb.com

## **CANINE SEMEN EXTENSION MEDIA**

**The purpose of the INTERNATIONAL CANINE SEMEN BANK (ICSB)<sup>™</sup> canine semen extension medium is threefold. This medium can:**

- 1. Increase the volume of low volume ejaculates to provide a greater likelihood of conception.**
- 2. Increase the general quality of the ejaculate by improving sperm motility and progressive movement, thereby giving the sperm cells a safe boost of energy and nutrients.**
- 3. Provide an excellent medium for extending the volume of the dog's ejaculate where the semen is to be divided for breeding multiple bitches at the same time.**

**In order to use the ICSB Extension Medium, it is necessary to keep the medium frozen until just prior to use. Thaw the medium, warm it to between room and body temperature (70 to 98 degrees F), and then add the desired amount to the collected semen sample from the dog to bring the volume to your required amount. If there is any remaining medium left in the vial, it can be reused if refrozen immediately. The media will remain viable for about 1 year if kept frozen.**

**If you have any questions, or wish to reorder, please contact ICSB at the address or phone number above.**

